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IDEALNY NA PREZENT !!!

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1. Alarm is initially turned off by default. Press and hold the power button to switch between ON/OFF. There will be a "Ding" alarm prompt after the alarm is turned on.

2. SpO2 indicates the alarm value setting of the pulse oximeter. Long press the power button to increase or decrease the alarm value. If the last value is higher or lower than the set value, there will be an alarm prompt.

3. PR indicates the alarm value setting of the pulse oximeter. Long press the power button to increase or decrease the alarm value. If the last value is higher or lower than the set value, there will be an alarm prompt.

4.2 Maintenance and Preservation

Replace the batteries timely when low voltage lamp is on.

Clean the surface of fingertip oximeter before it is used to diagnose patients.

Remove the batteries inside if you will not use it for a long time.

4.3 Product declaration

Resistance and manufacturer's declaration electromagnetic radiation

4.4 Possible problems and effective solutions

Problem	Possible reason	Solution
SpO2 or PR does not show accurately	1. Finger is not plugged correctly 2. Finger is too thick or too thin 3. Oximeter value is too low to be measured	1. Replug the finger 2. Try another finger 3. Please contact the local government for the product
SpO2 or PR is shown incorrectly	1. The finger might not be plugged deep enough 2. Finger is trembling or the patient is in movement status	1. Replug the finger 2. Please remain at rest
No Oximeter data can be turned on	1. The power is not turned on 2. Batteries might be installed incorrectly 3. The Oximeter might be damaged	1. Please replace the batteries 2. Please contact the local government for the product 3. Please contact the local government for the product
Indication lamp is suddenly off	1. The power is not turned on 2. Batteries might be installed incorrectly 3. The Oximeter might be damaged	1. Please replace the batteries 2. Please contact the local government for the product 3. Please contact the local government for the product